

## FR-45 First Responder Series Nitrile Exam Gloves

#### Features and Benefits

• Nitrile

• Exam Grade: 510K, CE & ARTG

• Powder Free

• Non-Sterile

• AQL 1.5

Chemotherapy Tested

• ASTM D6319, D6978

• EN 455 Parts 1, 2, 3

• Color: Blue

• Texture: Fingertip • Weight: 4.5 gram • Length: 240mm



Law Enforcement and Emergency Medical Responders are at high risk of being exposed to blood-borne pathogens and drugs. Therefore they need hand protection that provides a safe and strong barrier while maintaining dexterity and grip.

#### Usage













Responders

Industrial

Laboratory

Long Term

#### **Specifications**













Ordering Information				
ITEM #	SIZE	GLOVES PER BOX	BOXES PER CASE	GLOVES PER CASE
N45FRS	Small	100	10	1000
N45FRM	Medium	100	10	1000
N45FRL	Large	100	10	1000
N45FRXL	X-Large	100	10	1000

Email: contact@valiantshield.info



# FR-45 First Responder Series Nitrile Exam Gloves

### **Specifications**

OVERVIEW	
Material	Nitrile
Powder Content	Powder-Free
Freedom From Holes	1.5 AQL
Surface	Fingertip Textured
Weight (Based on Size M)	4.5 gram
Color	Blue
Cuff Length	240mm, Standard
Not Made With Natural Rubber Latex	Yes
Chemo Rated (Yes/No)	Yes, in accordance with FDA ASTM D6978
Food Grade Rated (Yes/No)	Yes, per CFR 177.2600

THICKNESS MEASUREMENTS	
Finger	0.114 mm (5 mil) +/- 0.05
Palm	.09mm (4 mil) +/- 0.05

PHYSICAL PROPERTIES			
Tensile	Before Aging	After Aging	
Force at break (N)	Spec ≥ 7 (Mean 9.5)	Spec ≥ 6 (Mean 9.9)	
Elongation at break (%)	Spec ≥ 500 (Mean 553)	Spec ≥ 400 (Mean 533)	

DIMENSIONS			
Item#	Size	Palm Width (mm)	Length
N45FRS	Small	85 ± 5	240mm / 9.5 inches
N45FRM	Medium	95 ± 5	240mm / 9.5 inches
N45FRL	Large	105 ± 5	240mm / 9.5 inches
N45FRXL	Extra-Large	115 ± 5	240mm / 9.5 inches

Email: contact@valiantshield.info



# FR-45 First Responder Series Nitrile Exam Gloves

Chemotherapy Drug	Permeation Drug Testing as	per ASTM 6978-05	
	BREAKTHROUGH	STEADY STATE	
<b>TEST CHEMICAL</b>	DETECTION TIME	<b>PERMEATION RATE</b>	
	MINUTES	(Avg) ug/cm²/min	
Dacarbazine	No Breakthrough detected	^	
	up to 240 minutes	0	
Carmustine	2.1		
Colomboanhamida	No Breakthrough detected	0	
Cyclophosphamide	up to 240 minutes	0	
Dovovskieje Ukrahlovida	No Breakthrough detected	0	
Doxorubicin Hychloride	up to 240 minutes	0	
F Florence and all	No Breakthrough detected	0	
5-Fluorouracil	up to 240 minutes	0	
Cianlatia	No Breakthrough detected	0	
Cisplatin	up to 240 minutes	0	
Ftomosido	No Breakthrough detected	0	
Etoposide	up to 240 minutes	U	
Decliteral (Toyol)	No Breakthrough detected	0	
Paclitaxel (Taxol)	up to 240 minutes	U	
Thio-Tepa	6.0		
Vincristine Sulfate	No Breakthrough detected		
vincristine Surfate	up to 240 minutes	0	
Methotrexate	No Breakthrough detected	0	
Methotrexate	up to 240 minutes	U	
NAitamoraia C	No Breakthrough detected	0	
Mitomycin C	up to 240 minutes	0	
Daitement	No Breakthrough detected	_	
Mitoxantrone	up to 240 minutes	0	
Ifactamida (Ifac)	No Breakthrough detected		
Ifosfamide (Ifex)	up to 240 minutes	0	
*Please Note that the following drugs have low permeation times			
Camms	ine 2.1 minutes, Thiotepa 6.0 mi	nutes	

Email: contact@valiantshield.info